



TEMRA[®]

I N T E R N A T I O N A L

TEMRA MASK 95+

PROTECTIVE MODEL PM01 & PM02

FFP2 / KN95



EN 149:2001+A1:2009



DIRECT CONTACT

+34 951 623 908
+34 644 077 930
info@temra.net
www.temra.net

MÁLAGA

Avenida Saenz de Tejada s/n
Edif. Fueng. Centro II N° 6-7
29640 Fuengirola
Málaga

MADRID

C/ Conde de Vilches N° 19
28028 Madrid
+34 91 356 50 75

MUNICH

Ismaninger Strasse 71 a
81675 München
+49 (0) 89 489 29869

FFP2

These are particle-filtering folded half-masks adapted to the wearer's face, mostly of white color.

These masks fall into the category of Personal Protective Equipment (PPE) according to EU regulation 2016/425 and EU directive: EN 149: 2001 + A1: 2009

Classification according to filtration efficiency and permeability:

FFP1 means a Particle Filtration Efficiency (PFE) equal to or higher than 80 percent and a maximum permeability equal to or less than 22%, which equates to an general protection level of at least 78%.

FFP2 means a Particle Filtration Efficiency (PFE) equal to or higher than 94 percent and a maximum permeability equal to or less than 8%, which equates to an general protection level of at least 92%.

FFP 3 means a Particle Filtration Efficiency (PFE) equal to or higher than 99 percent and a maximum permeability equal to or less than 2%, which equates to an general protection level of at least 98%

Test parameters: Paraffin oil and sodium chloride - 95l / min

DIRECT CONTACT

+34 951 623 908
+34 644 077 930
info@temra.net
www.temra.net

MÁLAGA

Avenida Saenz de Tejada s/n
Edif. Fueng. Centro II Nº 6-7
29640 Fuengirola
Málaga

MADRID

C/ Conde de Vilches Nº 19
28028 Madrid
+34 91 356 50 75

MUNICH

Ismaninger Strasse 71 a
81675 München
+49 (0) 89 489 29869

TEMRA MASK 95+

PROTECTIVE MODEL FFP2

- Reference: PM01 / JPM01 – no valve
- Reference: PM02 / JPM02 – with valve
- PMJ01 / PMJ02 – Junior masks
- Class: FFP2
- Bacterial filtration efficiency = 100%
- Particle filtration efficiency > 96%
- General protection level > 95%
- Material: Non-woven fabric, melt blown polypropylene, hot air cotton
- 5 layers for more protection than most conventional FFP2 masks
- 16cm x 10.5cm
- Conforms to CE standard according to EN149:2001+A1:2009 and PPE regulation

PM01



PM02



GALLERY – PM01 / PM02 - PMJ01 / PMJ02

PM01



PM02



PMJ01



PMJ02



PACKAGING – BOX PM01



PACKAGING - BOX PM02



TEMRA ACCESSORIES


FOR MODELS SM, SMM, PM, UPM03 & UPM04
OPTIONAL: PLEASE SPECIFY WHEN CONFIRMING
ORDER

LARGE SELECTION OF FASTENERS FOR ALL TEMRA MASKS:



- Reference: TMF01-02-03-04-05
- Face mask fastener
- Ear saver
- Material: PVC
- Adjustable mask fastener allows you to fasten your mask on your head instead of your ears

- Reference: TIS01
- Comfortable sponge
- TEMRA Mask Models SM, SMM and PM can all be equipped with easy wear sponges that makes the long wearing periods much more comfortable

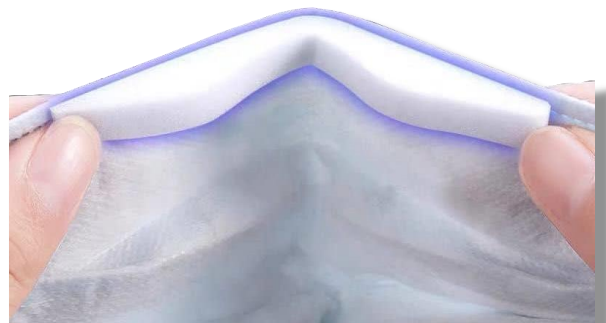

PACKAGING

1 Carton contains 100 boxes

1 Box PM01 contains 10 masks

1 Box PM02 contains 5 masks

Each mask is packed individually

Weight per carton: 10 KG

Units per carton PM01: 1000 masks

Units per carton PM02: 500 masks



PACKAGING

TEMRA MASK 95+ PM01/PMJ01 & PM02/PMJ02

- 1 Box PM01 contains 10 masks. 1 Box PM02 contains 5 masks (image 1)
- One English user information / instruction sheet per box (image 2)
- Every unit is packed in an individual plastic bag (image 3)
- Every plastic bag contains an official product certificate (image 4)
- Optional: According to order, each box will have an adjustable fastening per mask (image 5)



image 1



image 3



FFP2 - PROTECTIVE MASK

Product information and user guide

Language: German
 Product brand: Temra® International
 Product name: TEMRA MASK 95+
 Product model: PM01 (protective mask without valve)
 Classification: Type FFP2
 Product description: 150 x 105 mm, elastic ear bands, adjustable nose clip and five layers.
 EN 149: 2001 + A1: 2009
 EU Directive 2016/425 Annex 5 to personal
 Protective equipment (PPE)



Product information:

Manufacturer, importer and owner Product brand	Temra International Ltd The penthouse, Carolina Court, Giuseppe Cali Street, Ta'Xbiex XBX 1425
Production facility	Group of companies TEMRA® INTERNATIONAL Web: www.temra.net , Email: info@temra.net Wenzhou LK M. Technology Co., Ltd. Wenzhou National University Park No.89 Fengfang Rd., Ouhai District, Wenzhou, Zhejiang Province, China In cooperation With Temra International Ltd Group of companies TEMRA® INTERNATIONAL Web: www.temra.net , Email: info@temra.net
Authorized distributors:	Ayurveda Mijas SL Avenida Saenz de Tejada s / n Edif. Fuen. Centro II Nº 6-7, 29640, Fuengirola, Spain Temra International GmbH Karli-Lip-Road 35, 80992, Munich, Germany Magnaate Corpus Invest AS Drammenveien 130, 0277 Oslo, Norway

image 2



image 4



image 5

FFP2 - PROTECTIVE MASK

Product information and user guide

Language: English

Product brand: Temra® International

Product name: TEMRA MASK 95+

Product model: PM01 (protective mask without valve)

Classification: Type FFP2

Product description: 160 x 105 mm, elastic ear bands, adjustable nose clip and five layers.

EN 149: 2001 + A1: 2009

EU Directive 2016/425 Annex 5 to personal

Protective equipment (PPE)



Product information:

Manufacturer, importer and owner Product brand	Temra International Ltd The penthouse, Carolina Court, Giuseppe Cali Street, Ta'Xbiex XBX 1425 ----- Group of companies TEMRA® INTERNATIONAL Web : www.temra.net , Email: info@temra.net
Production facility	Wenzhou LK M. Technology Co., Ltd. Wenzhou National University Park No.89 Fengfang Rd., Ouhai District, Wenzhou, Zhejiang Province, China In cooperation With Temra International Ltd ----- Group of companies TEMRA® INTERNATIONAL Web: www.temra.net , Email: info@temra.net
Authorized distributors:	Ayurveda Mijas SL Avenida Saenz de Tejada s / n Edif. Fuen. Centro II Nº 6-7, 29640, Fuengirola, Spain Temra International GmbH Karl-Lip-Road 35, 80992, Munich, Germany Magnate Corpus Invest AS Drammenveien 130, 0277 Oslo, Norway

Description of the disposable or reusable product:	For single use, 5 layers: Layer 1: 30 g / m ² waterproof non-woven fabric Layer 2: Meltblown polypropylene 25 g / m ² with PFE Layer 3: hot air cotton 50 g / m ² Layer 4: Meltblown polypropylene 25 g / m ² with PFE Layer 5: 25 g / m ² skin-friendly non-woven fabric
Sterile or Not sterile:	Not sterile
Product features:	Five-layer face mask with elastic ear straps and adjustable nose clip.
Packing units:	10 piece per box
LOT:	specification on the box
Expiry Date:	36 Months to manufacturing date.
Warehouse- or Handling conditions:	Product should be stored in a clean, dry and ventilated area with a humidity of less than 80%.
WARNING:	The filtrationpower can deteriorate with long-term use. At Reuse exists the danger of a Cross contamination.
Performance according to requirements: EN 149: 2001 + A1: 2009 Ref .: UNIVERSAL CERTIFICATES 2163-PPE-972 Test report 2163-KKD-972	Bacteria filtration efficiency (BFE): = 100% Particle Filtration Efficiency (PFE):> 96% Total protection performance (GLS): > 95%



WARNINGS:

- Do not touch the outside of the mask. If so, wash your hands well.
- This mask is for single use only. Do not reuse.
- The mask must be replaced in good time. Long-term use is not recommended.
- It is recommended that you stop using the mouthpiece if you experience discomfort or side effects during use.
- The product is not washable. Please ensure that it is used within a valid time period.

Store in a dry, ventilated place away from open flames and combustible materials.

Change the mask if it is wet, short of breath, dirty or after 8 hours of use.

User Guide. How to make a mask touches down

1. Wash your hands with soap and water or hand sanitizer before touching the mask.
2. Remove a mask from its packaging, making sure that the mask does not have any obvious cracks, holes, or damage.

The side of the mask with a flexible and rigid edge faces up.

4. Ear strap mask: Hold the mask by the ear straps. Put a bow around each ear.
5. Pull the lower part of the mask down so that it covers your mouth and chin.
6. Shape the mask's noseband to match the shape of the nose for the best possible seal.



How to remove a mask

1. Wash your hands with soap and water or hand sanitizer before touching the mask. Do not touch the front mask. The face of the mask is contaminated. Only touch the clips / tapes / tapes. Follow the instructions below for the type of mask you are using.
2. Ear strap mask: Hold both ear straps and carefully lift and remove the mask. Dispose of the mask in a waste container according to your waste procedures. Wash your hands with soap and water or hand sanitizer.



Please contact us for more information:

Administration:

Barbara Kaszab

Email: info@temra.net

Tlf: + 34 632 909 942

Sales:

Manuel Sörries

Email: manuel@temra.net

Tlf: +34 662 116 012

www.temra.net

Avenida Saenz de Tejada s/n
Edif Fuengirola Centro II
29640, Fuengirola
Málaga
(Spain)



+34 951 623 908



+34 644 077 930



info@temra.net

DIRECT CONTACT

+34 951 623 908
+34 644 077 930
info@temra.net
www.temra.net

MÁLAGA

Avenida Saenz de Tejada s/n
Edif. Fueng. Centro II N° 6-7
29640 Fuengirola
Málaga

MADRID

C/ Conde de Vilches N° 19
28028 Madrid
+34 91 356 50 75

MUNICH

Ismaninger Strasse 71 a
81675 München
+49 (0) 89 489 29869

GALLERY – PM01 / PM02



GALLERY – PM01 / PM02



GALLERY – PM01 / PM02




GALLERY – JPM01 / JPM02




GALLERY – JPM01 / JPM02



GALLERY - BOXES

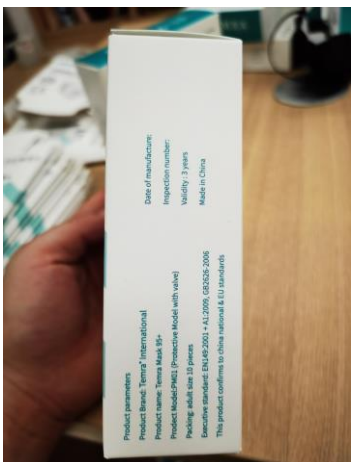

GALLERY - PRODUCTION

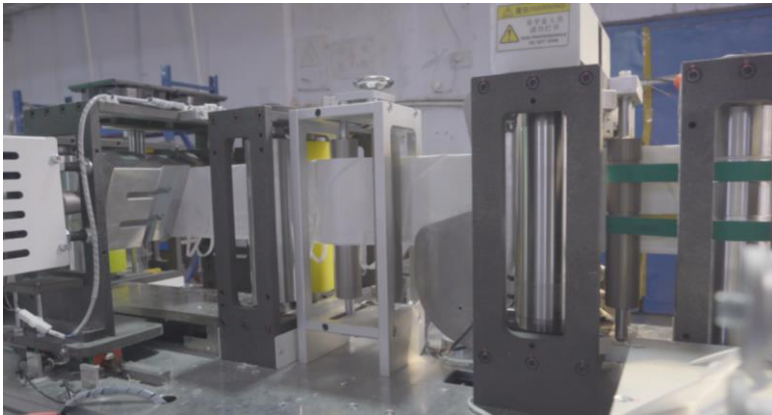

GALLERY - TESTING



GALLERY – TESTING



GALLERY - DELIVERY



DECLARATION OF CONFORMITY II



EU - DECLARATION OF CONFORMITY

according to the PPE regulation (EU) 2016/425

This declaration of conformity, issued under the sole responsibility of the producer, brand owner & importer, certifies that the following PRODUCTS

Product name: TEMRA MASK 95+
Description: Filtering half mask; foldable design (without valve)
Model: Protection model (PM01 / PMU01)
Standard: EN 149: 2001 + A1: 2009
Class: FFP2
Brand: Temra® International

Product name: TEMRA MASK 95+
Description: Filter end half mask; foldable design (with valve)
Model: Protection model (PM02 / PMU02)
Standard: EN 149: 2001 + A1: 2009
Class: FFP2
Brand: Temra® International

Manufacturing plant:
Wenzhou LK M. Technology Co., Ltd.
Wenzhou National University Park
No.89 Fengfang Rd., Ouhai District,
Wenzhou, Zhejiang Province, China
in cooperation with Temra International Ltd.
Group of TEMRA® INTERNATIONAL
info@temra.net - www.temra.net

Authorized distributors:
Temra International GmbH (Munich, Germany)
Ayurveda Mijas S.L. (Málaga, Spain)
Magnat Corpus Invest A.S. (Oslo, Norway)

COMPLY with the provisions following normative documents and conforms to essential requirements of relevant

Regulation (EU) 2016/425 for personal protective equipment (PPE) and harmonized standard EN 149: 2001 + A1: 2009

conform to the European Union,

including the basic health and safety requirements specified in Annex II of the PPE Regulation (EU) 2016/425.
The quality standard and design of the listed products are compatible with the reference articles named in the certificate N°2163-KKD-972, issued on 07/07/2020 by UNIVERSAL CERTIFICATION as the designated EU certification body N° 2163, Necip Fazıl Bulvarı Keyap Sitesi E2 block no.: 44/84 Yukarı Dudullu Ümraniye - İstanbul - Turkey. The products are also subject to the procedures specified in Module D of Regulation (EU) 2016/425, under the supervision of the Notified Body.

Date: September 18, 2020

Authorized by



Mr. Gerhard Peter Josef Wilhelm Wolfmeier
Temra International Ltd
The Penthouse, Carolina Court, Guiseppe Cali Street, Ta'Xbiex, 1425 XBX
TEMRA® INTERNATIONAL

DIRECT CONTACT

+34 951 623 908
+34 644 077 930
info@temra.net
www.temra.net

MÁLAGA

Avenida Saenz de Tejada s/n
Edif. Fueng. Centro II N° 6-7
29640 Fuengirola
Málaga

MADRID

C/ Conde de Vilches N° 19
28028 Madrid
+34 91 356 50 75

MUNICH

Ismaninger Strasse 71 a
81675 München
+49 (0) 89 489 29869

